



DNA Myself





Personalised Genetics

1. Remove the cap from the tube. Keep cap for later use.
Screw collection funnel on to the tube.

2. Spit in to the funnel until the level of the liquid in the tube, not including bubbles, reaches the 4ml line.
Delivery of saliva will take between 2 and 5 minutes.

3. Unscrew the collection funnel and replace the tube cap slightly. Discard the funnel.

4. Shake the collection Tube several times to mix the saliva with the solution.

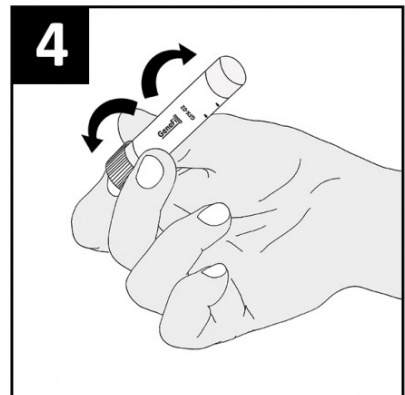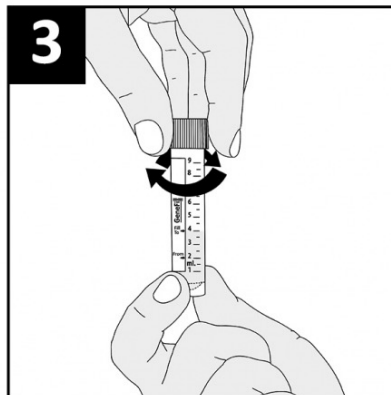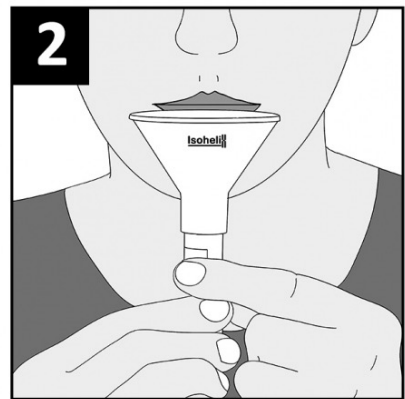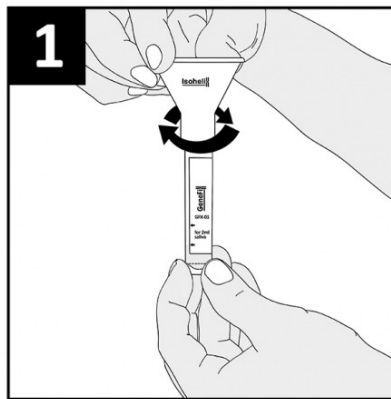

- Check fluid level is at 2ml line before opening.
- Do not eat, drink, smoke, brush your teeth or chew gum before 30 mins of use.